

INSTRUCTION FOR USE

DISPOCLAMP UMBILICAL CORD CLAMP

Material:

Polypropylene

Instructions for use:

- Place the umbilical cord clamp after delivery of fetus by placing two clamp in 4-5 cm distance to fetal abdomen.
- Press the clamp and close it with a quick and firm action, until it is completely blocked.
- The clamp is going to:
 - 1) block both the ends of the cord
 - 2) cut the cord itself
 - 3) separates in 2 parts: a baby-side clamp and a placenta-side clamp. one will stay with the baby and the second part will be attached to the placenta. The clamp can be removed when the cord is completely dry.
- The clamp is hold to stop bleeding from blood vessels.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions of infections.

Warning:

- To be use this product by a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instruction for use.

MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and /or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

Storage Condition :

- Do not expose to heat or direct sunlight

MD

Medical Device

REF

Reference No.

LOT

Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Sterilised By Ethylene Oxide
Single Sterile Barrier System



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damage



Caution



Consult Instructions For Use



Temperature Limit

CE
0123

EC REP

Authorised Representative
In The European Community



Disposafe Health and Life Care Limited

Plot No. 1&2, Phase-II, Sector- 59, Ballabgarh,
Faridabad-121004, Haryana (INDIA)

E-mail : info@disposafe.com

Web : www.disposafe.com

CUSTOMER CARE NO. : +91-129-4170201

EC REP

CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.

E-mail : office@cepartner4u.com

AW/IFU_240 ,REV:05, Dt. 18-02-2023