

INSTRUCTIONS FOR USE

MATERIAL USED:

- Polypropylene, PP, H.D.P.E & PVC TUBE (DEHP FREE)

INDICATIONS :

- To be used as an additional extension line during Infusion Therapy.

CONTRAINDICATIONS:

- Product should not be used in patients with known hypersensitivity to any of the materials used.
- Administration of highly viscous fluids.
- Not to be used for blood transfusion.

INSTRUCTIONS FOR USE :

- Peel open the pouch.
- Unscrew the protective cap from female luer.
- Connect male portion of I.V. Set/bag with female luer of tubing.
- Remove protective cap from male luer side of tubing.
- Hold male portion upright and open clamp of I.V. Set/bag and allow the solution to displace air entirely from tubing.
- Close clamp.
- Connect male portion of tubing to venipuncture device.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or

infections.

WARNINGS:

- To be use this product by a qualified doctor or a paramedic.
- Read instruction before use.
- The product should be used according to the instructions for use.
- **MANUFACTURE DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- The product should not be reprocessed.
- Visually inspect and carefully check the product and packing before use.
- Improper and handling may cause structural and or functional damage to device or packaging.
- The product is guaranteed sterile & non-pyrogenic if the package has not been opened or damaged.
- Do not clean or reesterilise. For single use only. Discard after use.
- Re-use of this device may change its mechanical or biological features and mau cause failure, allergic reactions or infections.
- Contains phthalate, should not be used for the treatment of children, pregnant women or nursing mothers.

REF

Reference no.

LOT

Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date

STERILE EO

Sterilised By Ethylene Oxide



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damaged



Caution



Latex Free

MD

Medical Device



Consult Instructions For Use



Temp. Limitation

EC REP

Authorised Representative
In The European Community



Keep away from sunlight