



## **INSTRUCTION FOR USE :-**

- Peel open the pouch.
- Unscrew the protective cap from female luer.
- Connect male portion of I.V. Set/ Bag with female luer of tubing
- remove protective cap from male luer side of tubing.
- Hold male portion upright and open clamp of I.V. Set/ Bag and allow the solution to displace air entirely from tubing.
- Close clamp.
- Connect male portion of tubing to venipuncture device.
- Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or infections.
- Should not be used for the treatment of children, pregnant women or nursing mothers.



## **CAUTION :**

- Do not use if package is open or damaged.
- Do not store at extreme temperature and humidity.
- Discard after single use.

AW/SB\_553, REV:06