

INJECTION STOPPER

MATERIAL USED :

ABS & Isoprene rubber

INDICATIONS :

- Injection of medicines
- For withdrawal of blood samples

CONTRAINDICATIONS:

- Not to be used for high viscous fluids and lipids administration.
- Product should not be used in patients with known hypersensitivity to the material used.

INSTRUCTIONS FOR USE:

- Check visually whether the package is intact.
- Open packaging and remove Injection Stopper from sterile package.
- Connect as desired to the female port of the infusion device in use
- Fill the injection with the medicine to be injected and inject the medicine from the needle less injection site
- Use only 21 G, 22G and 23G needle for injecting the medicine through the Injection Stopper
- After use of withdraw the injection from the Needle-less injection site and remove the injection stopper from the female port of the infusion device and after that connect the infusion set

WARNING:

- To be use this product is restricted to a qualified doctor or a paramedic.
- Read instructions before use.
- The product should be used according to the instruction for use.
- Manufacturer disclaims any responsibility for possible consequences resulting from improper use.**
- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and /or functional damage to device or packaging.
- The product is guaranteed non-toxic, sterile and Pyrogen free if the package is not opened or damaged.
- Do not clean or re-sterilize. For single use only. Discard after use.
- Re-use of this device may change its mechanical or biological features and may cause failure, allergic reactions or infections.

Storage Condition :

- Do not expose to heat or direct sunlight



Medical device



Reference No.



Lot No./Batch No.



Manufacturer



Sterilized By Ethylene Oxide
Single Sterile Barrier System



Date of Manufacturing



Use By / Expiry Date



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use if
Package is Damage



Caution



Temp. Limitation



Authorised Representative
in the European Community



Keep away from sunlight



Consult Instructions For Use



We Touch...Lives!

Disosafe Health and Life Care Limited

Plot No.1&2, Phase-II, Sector-59, Ballabhgarh-121004,
Faridabad, Haryana,India, E-mail:info@disosafe.com
Web : www.disosafe.com
Customer Care No. 91-129-4170201



CEpartner4U BV,
ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.
E-mail: office@cepartner4u.com

