

Karman-cath

KARMAN CANNULA

INSTRUCTIONS FOR USE

MATERIAL USED:

- HDPE, LDPE & PP

INDICATIONS:

- Intended use for the MTP specially, designed cannula for aseptic menstrual termination of pregnancy, distal end is coned with two large lateral eyes to facilitate the curettage, suitable for use with MTP syringe or suction apparatus, sterile.

CONTRAINDICATIONS:

- When used for uterine evacuation, vacuum aspiration is 98% effective in removing all uterine contents.
- Bladder stones.
- Retained products of conception require a second aspiration procedure.
- Abnormal uterine bleeding.
- Infertility.
- Infection of the urinary tract, kidney, or blood.

INSTRUCTION FOR USE:

- I.V. Cannula can be used to insert in a patient's vein order to give anaesthesia, medicine or other solutions for intermittent drug therapy/infusion or multiple fluid infusion.
- Intended use :All Karman cannula are intended for uterine aspiration/uterine evacuation in obstetrics and gynecology patients clinical indication for uterine aspiration are: treatment of incomplete abortion of uterine size up to 12 weeks from last menstrual period and endo manorial biopsy.

WARNING:

- The use of this product is restricted to a qualified doctor or a Paramedic.
- Read Instruction before use.
- All connections should be hand tightened, without over-tightening, to ensure their security and to prevent damage to the device.
- After use, this product may be a potential biohazard. Handle and dispose off in accordance with accepted medical practice and applicable local, state and country laws and regulations

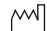
MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed sterile & Non-pyrogenic if the package has not been opened or damaged.
- Do not clean or resterilise. For single use only. Discard after use.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

REF REFERENCE NO.

LOT Lot No./Batch No.


 Manufacturer


 Date of Manufacturing


 Use By / Expiry Date


STERILE|EO Sterilised By Ethylene Oxide


 Do Not Reuse

 Do Not Resterilise

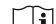
 Non-Pyrogenic

 Do Not Use If Package is Damaged

 Humidity

 Temperature

 Caution

 Consult Instructions For Use

EC REP Authorised Representative
In The European Community

 **Disosafe Health and Life Care Limited**
Plot No.1 & 2, Phase-II, Sector-59, Ballabhgarh,
Faridabad-121004, Haryana (INDIA)
E-mail : info@disosafe.com,
Web : www.disosafe.com
Customer Care No. 91-129-4170201

EC REP CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.
E-mail: office@cepartner4u.com

 0123